

AlphaTRAK²

BLOOD GLUCOSE
MONITORING SYSTEM

Accuracy Made Easy

Quick User's Guide



zoetis

AlphaTRAK 2 Meter Setup

Before performing a test, setup the volume, time and date on your meter. With the meter off, press and hold the **(m)** button for 3 seconds until S-E-T appears. Press **(m)** to bypass the reminder alarms.

Sound

To set the volume, press **(c)**



Press **(m)** to save your selection.

Time

To set the time:




Press **(c)** to change the hour and minutes.

Press **(m)** to save your selections.

Press **(m)** to accept time format or **(c)** to change.

Date

To set the date:

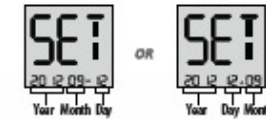

To change the month, press **(c)** to display the correct month and **(m)** to save it. Do the same for the day and year.

Press **(m)** to accept date format or **(c)** to change.

To save all your settings, press and hold down **(m)** for three seconds until the meter turns off.

Your meter is now set and ready to use. Follow the steps on the other side of the Quick User's Guide to perform a test.

For additional information, please contact Zoetis Inc.
Customer Service at: 1-888-963-8471

www.AlphaTRAKmeter.com
www.AlphaTRAKmeter.co.uk
www.AlphaTRAKmeter.ca

ART22860-001 Rev. A 09/15

Distributed by:
Zoetis Inc.
Kalamazoo, MI 49007

zoetis