


AlphaTRAK Lancing Device

LANCING DEVICE INSTRUCTIONS FOR USE

For use with AlphaTRAK Blood Glucose Monitoring System from Zoetis Inc. Intended Use: To obtain capillary blood samples for blood glucose testing. This lancing device is a simple, reliable way to acquire a drop of capillary blood for testing. It includes a system to adjust the depth of puncture to accommodate multiple testing sites.

CAUTION: To reduce the chance of infection:

- Never share alcohol or lancing device.
- Always use a new sterile lancet. Lancets are for single use only.

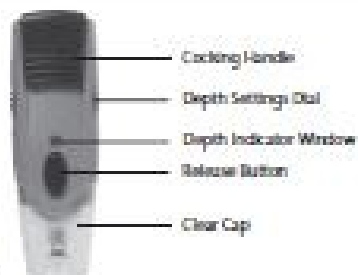

Contents:

(1) Lancing Device, (1) Clear Cap, (1) Instructions for Use

Read all instructions for the AlphaTRAK Blood Glucose Monitoring System and the lancing device carefully before using. For additional instructions consult your veterinarian/veterinary surgeon.

Lancing Device Instructions For Use:


To obtain a drop of blood:

1. Gather supplies needed (lancing device, lancet).
2. Wash hands with warm, soapy water. Dry thoroughly.
3. Snap the cap of the lancing device off at an angle.



4. Insert a new lancet firmly into the white lancet holder cap. (Pushing the lancet into the cap may crack the device, this is okay.)

Note: Do not reuse or re-use lancets.

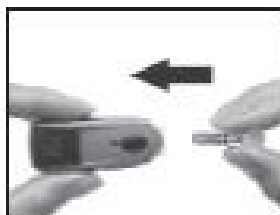

5. Hold the lancet firmly in place with one hand; use your other hand to twist off the rounded top.



6. Replace the cap until it snaps or clicks into place. Be careful not to touch the exposed needle on the lancet.



7. The lancing device offers four different settings. Level 1 is the shallowest depth. Level 4 is the deepest. Move the dial to the desired setting as shown in the depth indicator window.



8. The clear cap allows for better observation of the lancing site. Gentle pressure of the lancing device against the lancing site can be used to increase the puncture depth.

9. Pull the dark cocking handle out until it clicks. (You may have already cocked the handle in Step 4, this is okay.) You are now ready to perform a blood glucose test.




10. Prior to testing make sure the lancing site is clean and dry and select a comfortable and well-ill location for you and your pet.

11. Warming and massaging the test site will encourage good blood flow.

12. Hold the lancing device firmly on the clear cap against the site/ test lancing site. Push the release button to obtain blood drop. Applying continued pressure to the test site, may help with the formation of a blood drop.

13. Bring the AlphaTRAK meter and test strip to the blood drop. Gently touch only one side of the test strip to the blood sample. Ensure the sample touches the other side of the test strip in the black application area. Capillary action will pull the sample into the test strip.

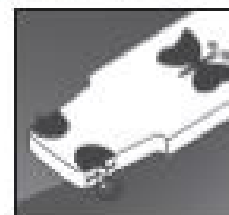

14. Snap the cap of the lancing device off at an angle.



15. To prevent an accidental needle stick, place the lancet safety cap on a hard surface. Carefully rock the lancet tip into the safety cap.

16. Push the release button to allow access to the white cap that holds the lancet. Push the white cap that holds the lancet and the lancet fall out. Do not pull the lancet straight out of the holder.



17. The rules for disposing of medical products and sharp objects vary so be sure you know what is appropriate in your area. Use and the used lancet properly in appropriate resistant container.

18. Replace the cap to prevent loss of cap.

Use isopropyl alcohol (available in the USA) or soap and water to wipe the outside of the lancing device. If you wish, remove the cap, wash it in warm water and rinse well.

Important: Do not immerse the body of the lancing device in water or any other liquid.

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a licensed veterinarian.

AlphaTRAK medical products are under the direction of a veterinarian/qualified veterinary customer service call 1-888-884-8838.

Available in:
Germany, UK, Spain,
France, Italy, Mexico,
Austria, etc. See 811115.

zoetis